

BPM-300 SOFTWARE REVISION HISTORY

7.1.4 07/01/21

- Use web-based tutorial; Compile option (%DIM) to make desktop dark if DIM is the default test background (in Utils|Misc.)

7.1.3 04/17/21

- Create CSV waveform Export as a UTF-8 file; Adjust left panel, 'Yes', 'No', 'OK', 'Cancel' and Protocol button widths for language differences.

7.1.2 01/20/21

- Correct initial menu state to %MF_GRAYED.

7.1.1 01/02/21

- Make additional dialogs and listboxes variable width for language variations.
- Increase Tooltip delay time; Use DB Connection (*.mdb, *.accdb) if available, otherwise use (*.mdb); Make Help|About 'OK' button the same size as others; Add %SS_NOPREFIX to Help|About|Translation label to display "&" correctly.
- Fix a Tooltip memory leak; Larger text size for tooltips; Use smaller text size for STATUS1 long messages; Adjust STATUS2, some button widths, and dialogs to better handle non-English languages; Add debug settings to show text sizes & Windows scale actor; Correct tooltip display in Reports|Print; Add special 'debug' tooltip to Help|About|OS Version.
- Use GRAPHIC TEXT SIZE statement rather than function; Add tooltip to Test|Protocol names.

7.1.0 08/19/20

- Major rewrite to handle Unicode character sets; Import previous version defaults if they exist; Show language in Utils|User.

7.0.9 07/20/20

- **NOT RELEASED**
- Make better use of hi-res displays; Increase Menu Text size; Correct display of background illumination value on 'demo' waveforms.

7.0.8 06/03/20

- Change Version format from x.x.x.x to x.x.x.
- Correct Protocol|Save checkbox function.
- Fix Utils|Misc ERG & VEP default Protocol settings.
- Remove ParPort references.
- Correct error message displayed when no BPM-300 waveforms are found in Utils|Import.

7.0.0.7 05/24/18

- Fix 'odd' Cancel button appearance in Utils|Import.
- Adjust Search and Cursor dialogs in Reports to accommodate Win 10 1803.
- Correct display of stimulus type "Grass" in Reports.

7.0.0.6 03/19/17

- Display proper Help for Utils (RGIHLP.chm only).
- Further correct Protocol|Save Bug.

7.0.0.5 10/29/16

- Correct Protocol|Save Bug.

7.0.0.4 09/17/16

- Add Windows version info to Help|About.
- Make default switch illum. 3.
- Remove Utils|Stimuli button & dialog.
- Adjust Reports|Search Age & Date control positions.
- Remove button status from STATUS2 in Test|Init.
- Update Tutor file to 300TUTOR.SWF.

7.0.0.3 05/28/16

- Remove %DS_MODALFRAME form main and help dialogs to correct failure of dialog headers to display in some Win10 systems.
- Improve screen filling for 3200x1800, etc. displays.

7.0.0.2 05/09/16

- Show file info during REPORTS|EXPORT.
- Tidy up MBDUMP.TXT format when appending data.
- Add Data Export Format selection to UTILS|MISC; Add CVS Data File Export option.
- Improve screen filling for smaller displays.
- Correct Test-Exit communication with hardware.
- Implement Switch Illumination.
- Use FREEFILE for disk I/O (missed two from 6.023).

7.0.0.1 12/25/15 -

- Correct various anomalies re. USB threading.

7.0.0.0 08/10/15

- Initial release (made from 6.023).