

## BPM-300 SOFTWARE REVISION HISTORY

### 7.0.0.7 05/24/18

- Fix 'odd' Cancel button appearance in Utils|Import.
- Adjust Search and Cursor dialogs in Reports to accommodate Win 10 1803.
- Correct display of stimulus type "Grass" in Reports.

### 7.0.0.6 03/19/17

- Display proper Help for Utils (RGIHLP.chm only).
- Further correct Protocol|Save Bug.

### 7.0.0.5 10/29/16

- Correct Protocol|Save Bug.

### 7.0.0.4 09/17/16

- Add Windows version info to Help|About.
- Make default switch illum. 3.
- Remove Utils|Stimuli button & dialog.
- Adjust Reports|Search Age & Date control positions.
- Remove button status from STATUS2 in Test|Init.
- Update Tutor file to 300TUTOR.SWF.

### 7.0.0.3 05/28/16

- Remove %DS\_MODALFRAME form main and help dialogs to correct failure of dialog headers to display in some Win10 systems.
- Improve screen filling for 3200x1800, etc. displays.

### 7.0.0.2 05/09/16

- Show file info during REPORTS|EXPORT.
- Tidy up MBDUMP.TXT format when appending data.
- Add Data Export Format selection to UTILS|MISC; Add CVS Data File Export option.
- Improve screen filling for smaller displays.
- Correct Test-Exit communication with hardware.
- Implement Switch Illumination.
- Use FREEFILE for disk I/O (missed two from 6.023).

### 7.0.0.1 12/25/15 -

- Correct various anomalies re. USB threading.

### 7.0.0.0 08/10/15

- Initial release (made from 6.023).